
# IQF OPERATING MANUAL

Integrated Quality Feedback

---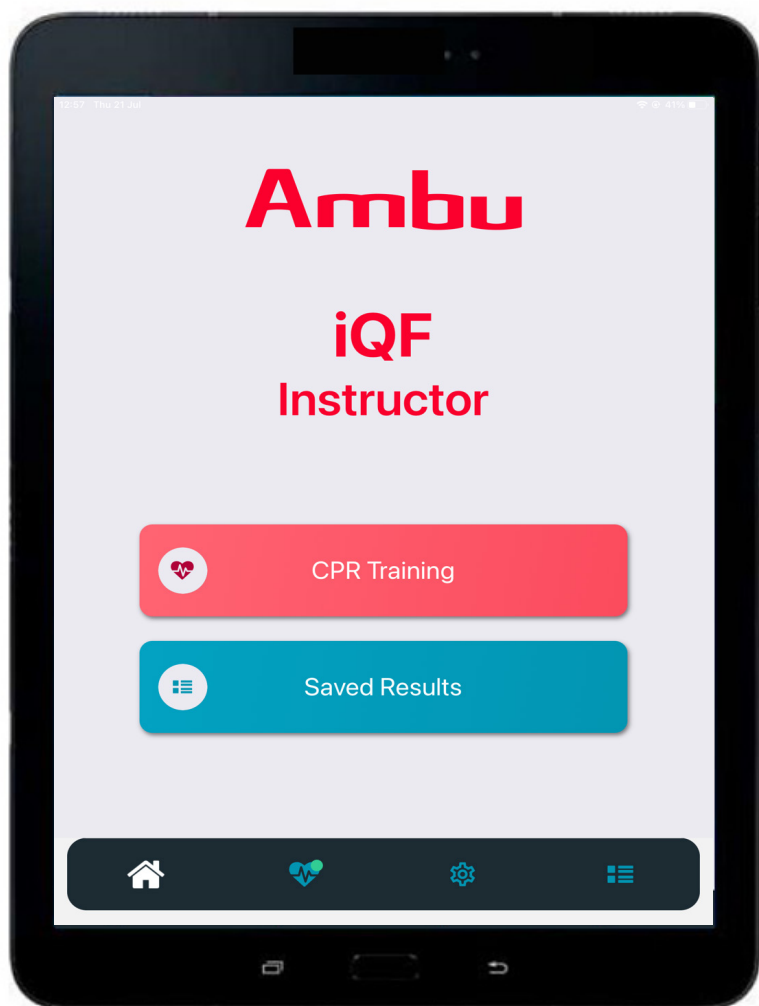

# Operating Manual iQF

<b>CONTENTS</b>	<b>PAGE</b>
English .....	3-15
Español .....	16-28
Deutsch (Bedienungsanleitung) .....	29-41
Italiano .....	42-54
Dansk .....	55-67
Français .....	68-80
Ελληνικά .....	81-93
Suomi .....	94-106
Magyar .....	107-119
Nederlands .....	120-132
Norsk .....	133-144
Polski .....	145-158
Português .....	159-171
Русский .....	172-184
Svenska .....	185-197
Türkçe .....	198-210
Česky (Návod k použití) .....	211-223
日本語 (使用法) .....	224-236
中文 (使用说明) .....	237-249

## English

<b>1. Introduction</b> .....	<b>4</b>
<b>2. Requirements</b> .....	<b>4</b>
2.1. Operating system .....	4
2.2. Technical Information .....	4
2.3. Connection Information .....	4
<b>3. Symbols and buttons</b> .....	<b>5</b>
<b>4. Getting Ready</b> .....	<b>7</b>
<b>5. Selection of CPR Training Parameters</b> .....	<b>7</b>
<b>6. Selection of Manikins</b> .....	<b>7</b>
<b>7. Starting a Session</b> .....	<b>8</b>
<b>8. Selection of Screen Type</b> .....	<b>9</b>
<b>9. Settings</b> .....	<b>10</b>
9.1 Settings - Instructor's App .....	10
9.2 Settings - Trainee's App .....	11
<b>10. Storage</b> .....	<b>12</b>
10.1 Storage - Instructor's App .....	12
10.2 Storage - Trainee's App .....	13
<b>11. Export and Print Out</b> .....	<b>13</b>
<b>12. Export, Import, Customize Guidelines</b> .....	<b>14</b>
<b>13. Updating the Firmware</b> .....	<b>14</b>

## 1. Introduction

The Ambu iQF App – Integrated Quality Feedback is an application for Android and for iOS that offers intelligent tools to improve the outcome of training with great precision, monitoring and documentation of the performance of infants and adults in cardiac arrest.

## 2. Requirements

### 2.1. Operating system

The apps are available for download on :

- The Google Play™ Store
- The Apple® App Store®

Minimum System requirements:

- Android™: Android OS version 8
- iOS™: iPhone 6, iOS 12

Recommended System requirements

- Android: Android OS version 10
- iOS: iOS 14.

### 2.2. Technical Information

- Three premium quality alkaline or rechargeable batteries size AAA (not included).
- Battery lifetime: 16h where manikin is connected to App sending sensor data.
- Operating Distance: The wireless connection has a range of approximately 10 meters outside having no obstacles in between.

### 2.3. Connection Information

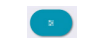
- Up to 6 manikins can be connected.
- Ambu Baby cannot be connected together with adult manikins in the same session
- When AmbuMan Compression is connected to other manikins, only compressions are displayed in the session.


### WARNINGS

1. In the event that you are going to keep your iQF manikin unused, it is advisable to remove the batteries for the best life of them.
2. When you change Screen Type, Guidelines, Duration and/or CPR Mode in the Setup, you always have to click on "Done", so the new setting is applied. If you only click elsewhere on the screen, it will go back to the last setting.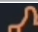


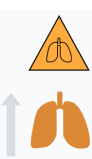
## 3. Symbols and buttons

The following section shows an overview the symbols and buttons, and gives further details on their function.

ICON	Meaning	ICON	Meaning
	Home		CPR Training
	Settings		Saved Results
	Quick Start Session		Manual Start
	Save Data		Continue
	Back		Switch Button
	Exit Session		Live Results
	Pause		Stop Recording
	Metronome		zoom in/out
	Session Joined		More details
	Manikin connected or disconnected		Bluetooth activated or not
	Edit name/field/ Guideline		Manikin connected
	PDF		Exit
	Delete		Filter
	Manikins		guidelines
	Battery Status		Update Firmware
	Factory Reset		Export
	Import		New Custom
	Save the session		Print
	Unfold options		Confirm
	Manikin type locked		Add (Name, Logo...)

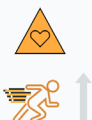ICON	Meaning	ICON	Meaning
	Date Range		Clear
	Folder Item		Select multiple items
	Collapsed Folder		Open Folder
	Duration		Unlimited
	Score (60%-75%)		Score (>=>75%)
	Return to Overview		Score (<60%)
	Open Menu		Frequency info

#### Perfusion Warning Icons

	Compression not deep enough		Compression too deep
	Compression frequency too fast		Compression frequency too slow
	Ventilation volume too high		Ventilation volume too low
	Leaning detected		Lungs - Perfusion
			Heart-Perfusion


## 4. Getting Ready

1. Prepare your manikin:
  - Open the lid
  - Insert three AAA batteries
  - Turn on the manikin using the on/off lever

2. Prepare your device:
  - Activate the Bluetooth
  - Activate the location

## 5. Selection of CPR Training Parameters

Click on the instructor's app.  
Click on "CPR Training" and find the following screen with four tabs to configure the desired session. Click on each one to select your choice and click on "Done" once finished.  
Let's see what we have on each of the tabs: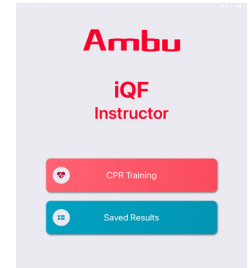

### - Click on "Screen Type"

You can choose between the following: Heart/lung, Bar graph, Perfusion, Competition with runners and Competition with hearts

### - Selection of the Guidelines

You have the possibility to choose between: ERC 2020 Adults, ERC 2020 Infants, AHA 2020 Adults, AHA 2020 Infants.


### - Selection of "Duration"

You can select minutes and seconds and also unlimited session by clicking on "Switch" button.

### - Selection of "CPR Mode"

You can choose the following options:

- 1 person CPR
- 2 person CPR
- Compression only
- Ventilation only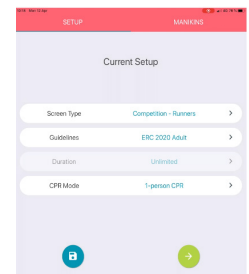


## 6. Selection of Manikins

Once the CPR parameters have been selected, click on the green arrow button to move to the manikin selection or click on the tab "Manikins" in the upper right. You will see all available manikins. If more than one type of manikins is connected, you can **activate a filter** to only show the required manikin type.

**To select the manikin**, click on it. The outline of the manikin will change to blue color and the battery state is visible. If you want **to unselect** click on the manikin and the outline will change from blue to black.

If you want **to add trainee names**, click on hold on manikin, a field will be shown where you can enter the names.  
Click on the green tick button.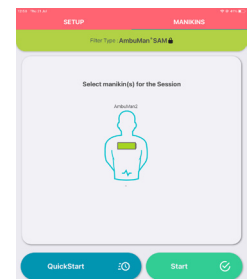

## 7. Starting a Session

### 7.1 Starting a Session - Instructor's App

Once the manikins were selected, you can use two options to start a session:

- **Quick Start:** the session will immediately start recording.
- **Start button:** the session will open but the section needs to be started manually.

Click on it and will appear a message "Creating Session".

**In the overview** you can see all manikins connected to the sections, showing the screen type that was selected.

**To see the details** of the session, click on screen of the manikin.

Click to the **back button** to return to the overview.

If you click on **pause**, the live performance will still be available continue but not recorded.

The **metronome** can be used here by clicking on the metronome symbol in the bottom.

#### 7.1.1 Results and Detailed Results

When the session is finished or stopped manually you can see the **overview of the results**.





To see the details, **click on the session**. To see the graph, **click on the time series**.

You can scroll through the graph and change the **time scale**.

**To see more details**, click on detail button. Click on the back button to go back to the overview.

Each session can be saved, converted into a PDF, or just exit the session by clicking on the three dots in the upper right.

In the overview you can save all sessions using the disk symbol or you can exit the session.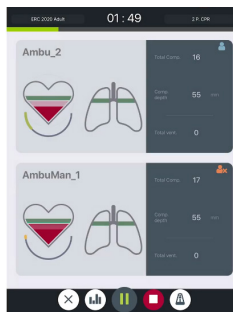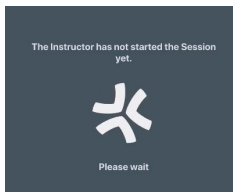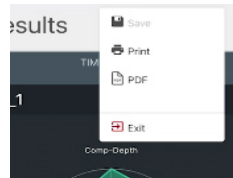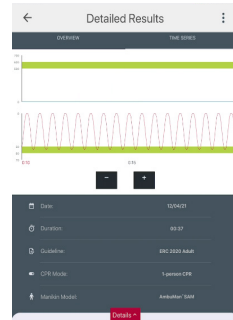

### 7.2 Starting a Session - Trainee's App

Once clicked on the app symbol, click on "CPR Training". The trainee can choose which **screen type** to be displayed and click on "Done" once chosen.

- Heart / Lung
- Line Graph
- Bar Graph
- Perfusion

To continue with the selection of the manikin click on the green button.

The trainee can type his/her name and click on "OK".


After selecting the screen type, select which manikin to run the session on. And then click on "Join Session".

If the instructor has not started yet the training, the trainee will see this image and has to wait.

If the student successfully connected to the manikin, the icon in the top right corner will change from orange to blue, letting the instructor know.

Once the instructor has started the session, the trainee app will automatically and start the CPR training.

Warning signs will appear if the training is done incorrectly.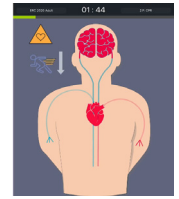

## 8. Selection of Screen Type

Once you have entered the app, click on "CPR Training" and find the following screen with four tabs to configure the desired session.


Click on "Screen Type" Scroll through the wheel to select the Screen type.

Let's see what we have on each of the tabs:

#### - Heart/lung:

Thanks to the lungs you can see the performance during ventilaton and thanks to the heart you can visually see the compressions


On the right side you will see the total amount of compressions and ventilations. At the bottom, you will find the metronome funtion to measure the frequency of compressions.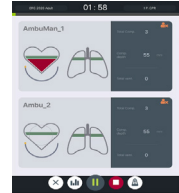

#### - Bar graph

In this screen type you will see the performance through bars.


You see the single compression and ventilation together with the right volum of the compression depth.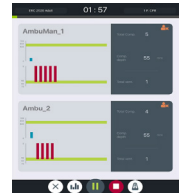

#### - Line graph


There are two graphs, one is for ventilations anthe other one for compressions. In this screen type you don't have a speedy meter showing the frequency of compression.

Here you also have the metronome and on the right side you will see the amount of compression and ventilations.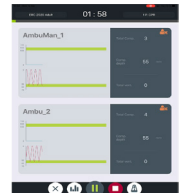

#### - Perfusion


If you do good compression performance at the right frequency and right depth, you can see how the circulation is built up.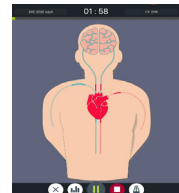

#### - Competition with runners and Competition with hearts

If you want to teach and learn in a fun way then use the "competition" option. You can compete with runners and hearts.

After choosing the Screen Type, an specific guideline, the duration of the session and the CPR mode, you are ready to start the session.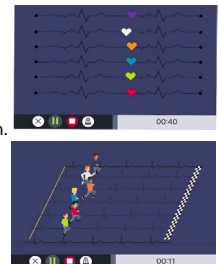


## 9. Settings

### 9.1 Settings - Instructor's App


To access the settings, click on the set-up button at the bottom of the screen

Here you see the overview of the six tabs setting available.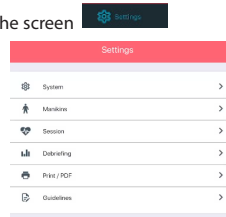

Click on **"System"** in the Settings.


You can change the language by clicking on the dropdown under **"Configuration"**. You can select or deselect if the manikin should enable automatic data.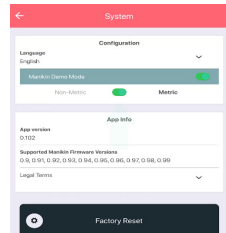

You can also choose if you want to have **"Metric or non Metric"** display.

If a factory reset is needed, then click on the black button in the bottom of the screen.


Click on **"Manikins"** Here all connected manikins are shown, to enter click on **"Inspect"**. You will find the name, the type, ID and software. The name can be changed by clicking in the name field. Click enter once typed.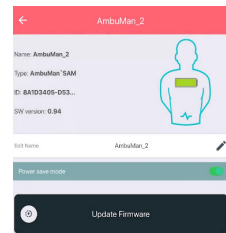

Power safe mode is available, this means the manikin will be turned off after 30 mins without use.

Click on **"Update Firmware"** to automatically update the manikin to the latest firmware. A notification will appear, stating the firmware was successfully updated. Disconnect the manikin and restart.


Click on **"Session"**. To configure the session. The main screen here shows the overview of a session. Up to four values can be selected.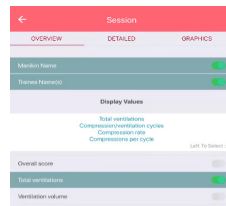

The **"Detailed"** view allows you to select the set of values that are available during the session without limitations.

If you are having a line graph or bar graph diagram, you can go to the third tab **"Graphics"** where you can change the interval of the screen.

Click on **"Debriefing"**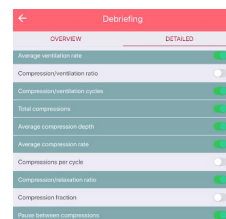

To select which values you want to be shown.

The **"Detailed"** view allows you to select the set of values that are available during the session without limitations.

Click on **"Session"**. To configure the session. The main screen here shows the overview of a session. Up to four values can be selected.


The **"Detailed"** view allows you to select the set of values that are available during the session without limitations.

If you are having a line graph or bar graph diagram, you can go to the third tab **"Graphics"** where you can change the interval of the screen.

Click on "Guidelines". To see the guidelines that are currently installed.

If you want to view them, click on them and if you want to modify something, just click. Change the value, confirm with OK and press the Disk button to store it.


The guidelines can also be exported, imported and customized by clicking on the desired option at the bottom.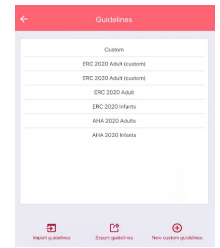

### 9.2 Settings - Trainee's App

To access the setting click on the set-up button in the upper right corner of the screen. Here you see the overview of the four tabs setting available.

Here the overview of the four tabs setting is displayed.
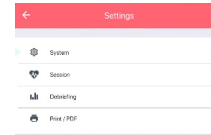


Click on **"System Settings"**

If the trainee wants to change the language by clicking on the dropdown.

They can also choose if they want to have "Metric or non Metric" values.

If a factory reset is needed then click on the black button in the bottom of the screen.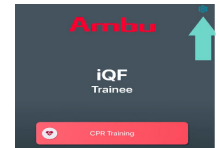
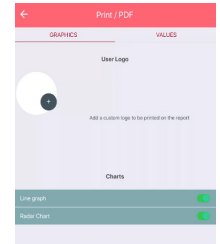


Click on **"Session"**

To configure the session. The main screen here shows the overview of a session. Up to four values can be selected.

The **"Detailed"** view allows you to select the set of values that are available during the session without limitations.


If you are having a line graph or bar graph diagram, you can go to the third tab "Graphics" where you can change the interval of the screen.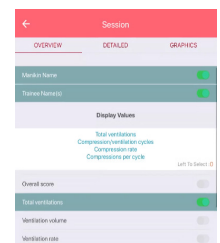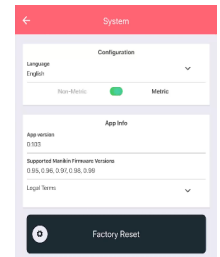

Click on "Debriefing"

To select which values you want to have shown.

The "Detailed" view allows you to select the set of values that are available during the session without limitations.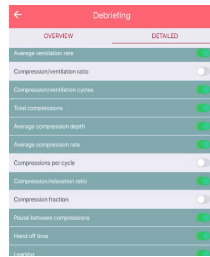


Click on "Print / PDF"

To decide how you want the printout to look.

Here you can enable or disable the Line Graph or the Radar Graph.

If you click on the "Values" section you can decide which values should be shown in the printout or the PDF.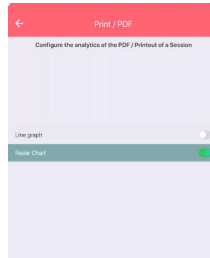

## 10. Storage

### 10.1 Storage - Instructor's App

To access to the storage, click on "Storage" on the lower tak bar or "Saved Results". Here you see all the stored sessions.

#### Saved Sessions - Options

##### Filter button

If you want to search for specific sessions click on the "Filter" button. Once filters were chosen, click on "Apply Filter".

##### Structuring Folders

Make your life easier by structuring sessions using folders by clicking on the three dots in the top right and select "New folder". Type the name and click on "Enter".



Folders can be deleted by pressing down on them and clicking on the bin button.

If you want to have session moved to a specific folder press and hold the tab of the session and click on "Move to folder" button.

##### Deleting sessions

If you want to delete a session press down on the session until it is marked and click on the "bin" button.

If you want to view the details of a session, tap on it and the details will be opened. Here you find two taps: Overview and Time Series. On either of them, you can click on the red "Details" button if you want more details.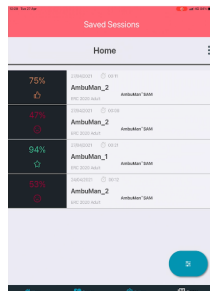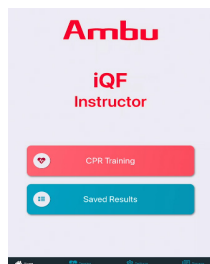


## 10.2 Storage - Trainee's App

To access the storage, click on "Saved Results". Here you see all the stored sessions.

Session can be deleted by clicking on the bin button. Confirm the deletion by clicking "Delete" 



If you want to view the details of a session, tap on it and the details will be opened. Here you find two taps: Overview and Time Series. The Time Series tab features a graph in which you can change the time scale.

In either of them you can click on the red "Details" button at the bottom of the screen, if you want to have more details.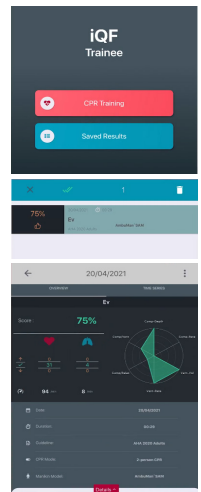

## 11. Export and Print Out

To access the storage, click on "Storage" on the lower tak bar or on "Saved Results". Here you see all the stored sessions. There you get your stored session, click on the one you would like to work on and see the results. Click on the three dots in the right corner of the screen.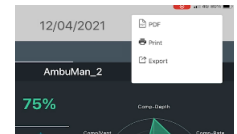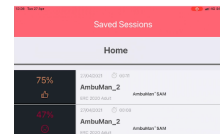

##### Click on "PDF"

To convert the session into PDF and enter the file name by clicking on the three dots in the upper right corner. The PDF will be saved on your device.

##### Click on "Export"

To export the session results on the device and enter the file name.

- On iOS Device: open "Files", choose the folder "Ambu iQF Instructor"> Session Data



- On Android devices: you can locate exported session by clicking on Internal storage > AmbuIQF files > Session Data

##### Click on "Print"

To print the session results. Choose the print setup of your choice.

- On iOS Device: open "Files", choose the folder "Ambu iQF Instructor"> Session Results

- On Android devices: you can locate exported session by clicking on Internal storage > AmbuIQF files > Session Results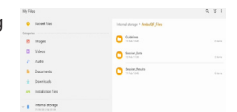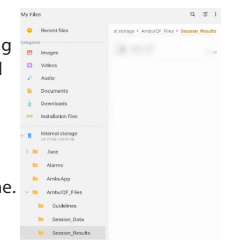

## 12. Export, Import, Customize Guidelines

To access the setting click on the set-up in the bottom of the screen. Here you see the overview of the six tabs setting available.



Click on **"Guidelines"**

To see the guidelines that are currently installed on the manikin.


If you want to view them, click on them.

Here you see the overview of all values that are set in the guidelines. If you want to modify something, just click inside. Change the value, confirm with OK and press the Disk button to store it.

The guidelines can also be exported, imported and customized by clicking on the desired option at the bottom.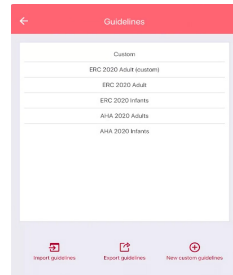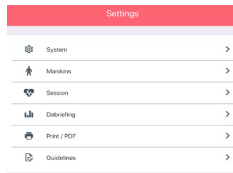


Click on **"Export Guidelines"** to export the guideline of your choice and click on the tick green button.

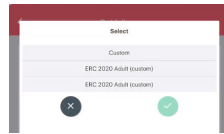Click on **"Import Guidelines"** to import the guideline of your choice. The system will ask you for the location of the document. Select it and click on "Done"

Click on **"New Custom Guidelines"** if you want to create a new guideline of your choice. Here you can enter all the values you wish. And confirm by clicking on the "save" button.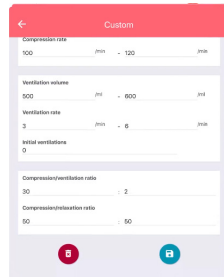

## 13. Updating the Firmware

To access the setting click on the set-up in the bottom of the screen. Here you see the overview of the six taps setting available.




Click on **"Manikins"**

Here all connected manikins are shown, to enter click on "Inspect". you will find the name, the type, ID and software. The name can be changed by clicking in the name field. Click enter once typed.

**Power safe** mode is available, this means the manikin will be turned off after 30 mins without use.

Click on "Update Firmware" to automatically update the manikin to the latest firmware.

The updating process can be followed in the percentage.

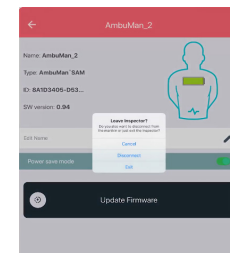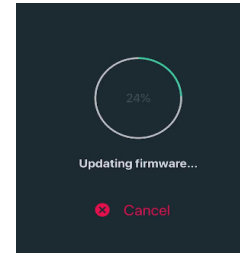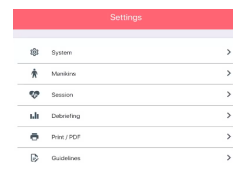Once the update has finished, you will get a notification stating the firmware has been successfully updated.

Follow the instructions and disconnect the manikin and restart.

If you want to **leave the inspection**, press either "Disconnect" to remove the connection between the manikin and the device or just click on "Exit".

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Ambu A/S